


MEDTEQ

Precision Pressure Sensor

Models
PPS 4.0 / PPS 3.0

Operation Manual

For use with software versions 4.0.0.x / 3.2.0.x

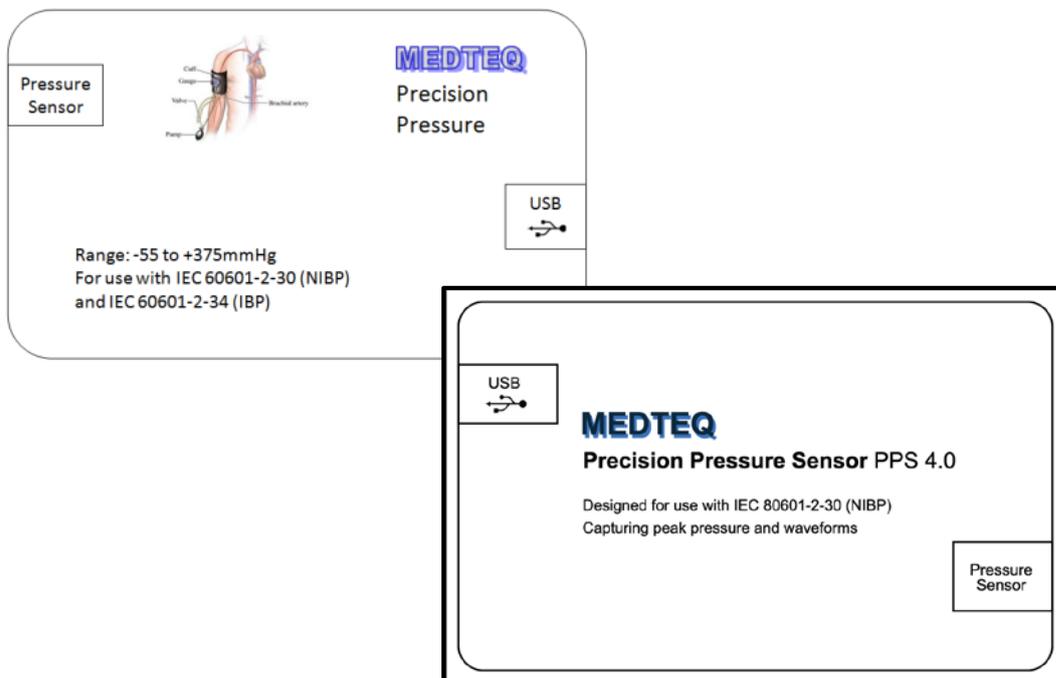

MEDTEQ
545-56 Tsujikuru-cho, Ise-shi, Mie, Japan
Ph (+81) 090-9897-2340
www.medteq.info



This page is intentionally blank

Contents

1	Introduction	4
2	Description	5
3	Installation	6
3.1	Software installation	6
3.2	USB driver	6
4	Set up and use	6
4.1	Pressure sensor	6
4.2	USB Module, PC software	7
4.3	PC screen	7
5	Specifications	8
6	Calibration	9
7	Contact details	9

1 Introduction

The MEDTEQ Precision Pressure equipment is a PC based USB module, designed to capture fast changing dynamic values associated with testing of medical electrical equipment, and to provide graphs of pressure with time for detailed analysis. For testing automated NIBP equipment against the single fault requirements in IEC 60601-2-30 / IEC 80601-2-30, the software automatically records the timing of events as necessary to determine compliance.

The system uses a standard IBP sensor which has been found to be easily connected to a variety of pressure systems in medical equipment. The sensor can handle both wet and dry conditions¹. The IBP sensor is combined with precision electronics, non-linear curve fitting to provide a convenient sensor with fast response time, high accuracy with software interface.

The range of pressures can be selected by the client up to $\pm 300\text{kPa}$ ².

Currently three versions are available:


Range	Intended purpose	Features
-50 to +220kPa	IEC 60601-2-24, infusion pumps	<ul style="list-style-type: none"> Connects directly to infusion sets, works with fluids Software records peak pressures range assumes limit of 200kPa should not be exceeded (the limit for infusion sets per ISO standards) response time 80ms moderate accuracy ($\pm 0.5\text{kPa}$, $\pm 0.2\%$ full scale)
-50 to +400mmHg	IEC 80601-2-30 NIBPs	<ul style="list-style-type: none"> IBP sensor found easy to connect to air circuits Records peak pressures range takes into account limit 330mmHg high accuracy ($\pm 0.5\text{mmHg}$ $\pm 0.2\%$ full scale) response time 80ms software automatically records timing at thresholds required in the standard in an event log, and informs if limits in the standard may have been exceeded
Dual range -50~+400mmHg and -50~+200kPa	As above	<ul style="list-style-type: none"> As above. Incorporates range switching to enable both applications in a single unit Compatible with software version 4.0.0.x

¹ For dry conditions, the sensor should have all fluid removed, if necessary by allowing to dry.

² Currently MEDTEQ's calibrated reference equipment is limited to -50 to 200kPa. Reference equipment is necessary to calibrate the sensor and provide non-linear compensation.

2 Description

The MEDTEQ Precision Pressure system consists of a standard IBP pressure sensor, a USB module and a PC for displaying the current pressure, peak pressure, zeroing function, graph and event logs:

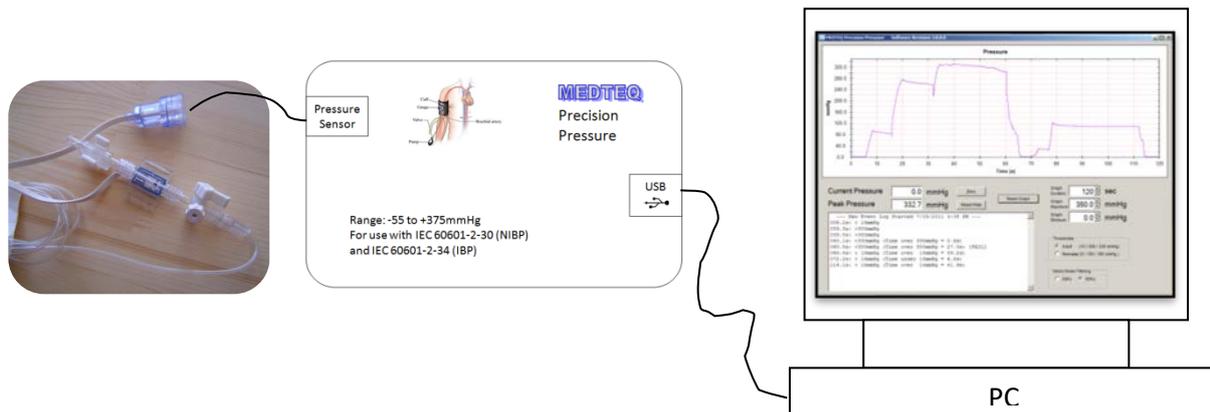

The sensor is a standard bridge type of sensor used for IBP measurements ($5\mu\text{V}/\text{V}/\text{mmHg}$). The USB module contains special circuitry to ensure high accuracy measurements can be derived from such a sensor, and is combined with non-linear compensation in the PC software.

As with all pressure sensors, the zero point (output with zero pressure applied) can drift with time. As such the equipment is provided with a "Zero" button which allows this offset to be removed. The zero button can also be used to remove offsets from a head of water (if used with liquid circuits). The zero offset at power on with dried, air filled sensor with no pressure applied should not exceed $\pm 3\text{mmHg}$ ($\pm 0.5\text{kPa}$).

The sensor itself has a very high frequency response, and is sampled at 1kHz^3 . To removed mains noise, the signal is then integrated over 40ms. This means that the system can take up to 80ms to fully respond to a change in pressure.

The sensor is sensitive to temperature, with a co-efficient of around $0.025\%/^{\circ}\text{C}$. The accuracy specifications are intended for a normal laboratory environment of $20\text{--}30^{\circ}\text{C}$. Use outside of this range may exceed the specifications at the full range pressure (e.g. at 400mmHg , the accuracy may be $\pm 1\text{mmHg}$ at 35°C instead of $\pm 0.5\text{mmHg}$).

³ This value is for 50Hz, which is worst case. For 60Hz environment, the sample rate is 1200Hz, leading to a response time of 67ms.

3 Installation

3.1 Software installation

The MEDTEQ IBP Simulator system uses a normal PC to interface and control the USB module. The PC should meet the following requirements:

- Windows PC (XP or later)
- Microsoft .NET 2.0 or higher
- Administrator access (if necessary for installation of software/driver)
- Free USB port
- Minimum 512MB RAM

Software is available from the MEDTEQ website:

<http://www.medteq.info/med/PrecisionPressureSoft>

Three options are provided which should be used in order of preference. Follow the instructions on the website.

3.2 USB driver

The system uses a USB mode called “CDC” which emulates a serial COM port for which Microsoft Windows® already has the driver for this installed. However, it is necessary to link the USB Module to this driver, which follows a process similar to installing a driver.

The linking file “mchpcdc.inf” is available at <http://www.medteq.info/medteqsoft/mchpcdc.inf>. Copy this file to a known folder. When the USB is first connected, select manual installation, and point to folder containing the above linking file. Continue to follow instruction. There may be a warning that the driver is not recognized by Windows® which can be ignored. This linking file is provided by Microchip® for use with PIC microprocessors having in built USB function. Administrator access for the PC may be required.

4 Set up and use

4.1 Pressure sensor

The pressure sensor connects to the USB module using connector as marked.

If being used with water, ensure the sensor is maintained in a firmly fixed position and orientation. Moving the sensor around changes “head of pressure” and yields inconsistent results.

If being used with air, ensure that all water is removed, ideally allowing the sensor to dry out overnight before use. Small drops of water on the sensor can move with air and cause variations of $\pm 0.3\text{mmHg}$.


The sensor should be maintained in a laboratory environment of $25 \pm 5^\circ\text{C}$ for specified accuracy. For a range of $15\text{-}35^\circ\text{C}$, double the specified accuracy.

4.2 USB Module, PC software

Connect the USB module to a free USB port, and after allowing time for the PC to recognize the USB device, start the MEDTEQ software for the IBP simulator (IBP Sim).

4.3 PC screen

The following screen shot is for Version 4.0.0.0

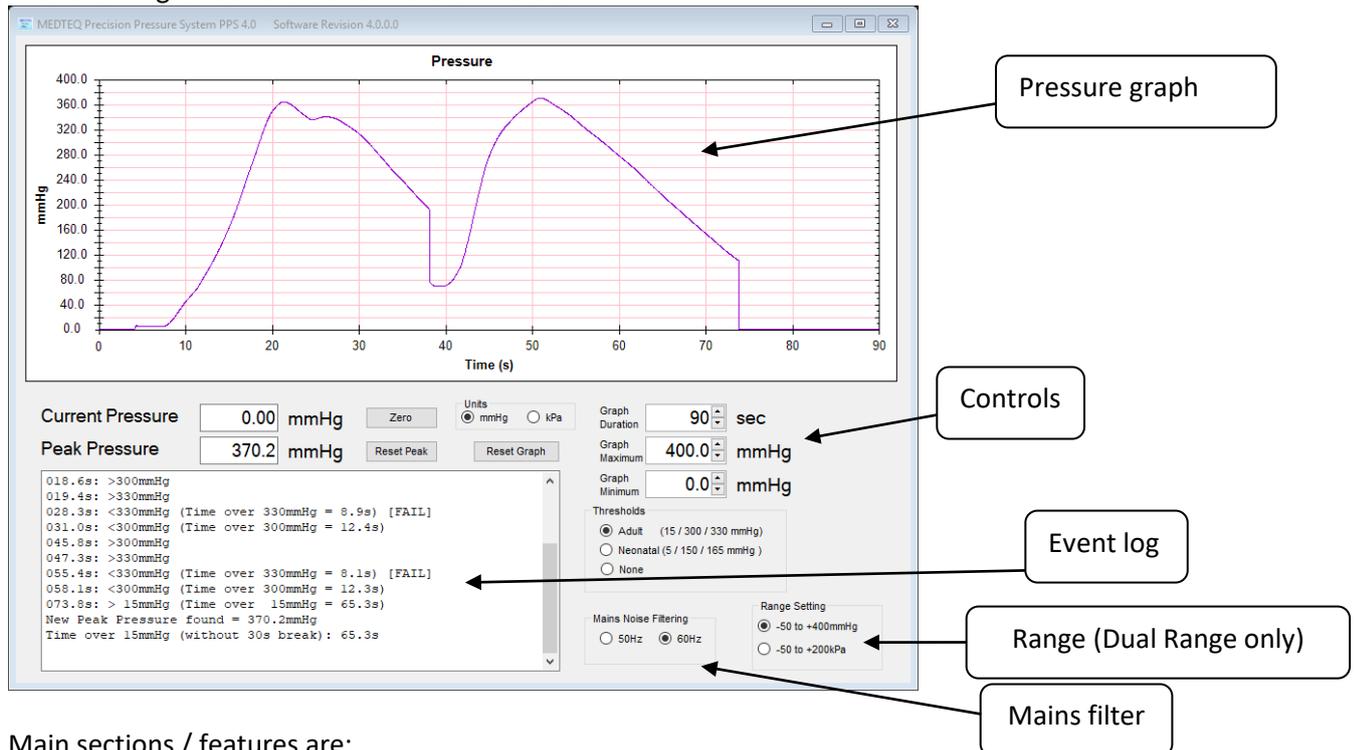

Main sections / features are:

Pressure graph	Graphical display of the pressure. Can be adjusted using the following controls: <ul style="list-style-type: none"> • Graph duration • Graph maximum • Graph minimum • Graph reset (clears the graph and also the event log) <p>The graph can be copied to clipboard for inclusion in test reports, and has functions such as the ability to zoom in or inspect individual points.</p>
Current pressure display	This records the current (real time) pressure measured by the sensor
Peak pressure display	The keeps the maximum pressure recorded. The peak can be reset at anytime using the “Reset Peak” button.
Zero button	This button sets the zero point to the current pressure at the time the button is pressed. It is intended to be used to adjust for long term drift, and the head of water in tests with liquids.
Units	Units can be selected as either mmHg or kPa. Note: currently the event log and thresholds use mmHg, irrespective of the units setting. The events will still be accurately recorded, for example if kPa is selected, with adult threshold limits, the event log will record “300mmHg exceeded” when the pressure exceeds 40kPa.

Event log	<p>The event log is used to record the timing of events at certain thresholds and duration over thresholds as associated with IEC 60601-2-30 (IEC 80601-2-30), for example (adult thresholds):</p> <ul style="list-style-type: none"> • Time when pressure goes over or under 15mmHg • Time when pressure goes over 300mmHg • Duration that pressure exceeds 330mmHg • Duration that pressure exceeds 15mmHg <p>A new event log is started when the “Graph Reset” button is pressed. However the previous event log remains. The event log is a simple text field which can accept the usual text operations (cut, copy, paste, delete, typing).</p>
Thresholds	This allows the user to set the thresholds for either adult or neonatal reflecting the limits in the standard (IEC 60601-2-30, IEC 80601-2-30).
Range	For dual range only, this switches between the two ranges.
Mains filter	Set to 50Hz or 60Hz depending on the environment.

5 Dual range equipment

Equipment fitted with the dual range function has hardware switches to modify the range and offset presented to the system. The Range Setting can be used switch between ranges.

Notes:

- during range switching, the pressure will temporarily read zero.
- for special request ranges, a new version of PC software may be needed.

6 Specifications

Parameter	Specification	Notes
Range	Up to ± 300 kPa ($\pm 2,200$ mmHg)	Depends on the requirements of the user
Sensor	Standard IBP ($5\mu\text{V}/\text{V}/\text{mmHg}$)	
Accuracy	Up to $\pm 0.2\%$ full scale	Full scale = (max – min) pressure Requires non-linear compensation Within $\pm 5^\circ\text{C}$ of calibration temperature
Response time	80ms	40ms software averaging applied
Power supply	USB, +5Vdc, 0.2A	

7 Calibration

Each unit is individually adjusted using non-linear calibration against 0.05% reference equipment, with the calibration coefficients stored in firmware of the individual unit. Calibration is performed in production using traceable equipment with a report provided. A copy of the ISO 17025 certificate for the reference equipment can also be provided (may be in Japanese).

While MEDTEQ calibration is traceable it is not issued under any accreditation scheme. Users that require third party ISO 17025 accredited calibration should arrange on their own or contact MEDTEQ to quote and arrange prior to shipping.

8 Contact details

MEDTEQ can be contacted by the following means:

Email: equipment@medteq.info or peter.selvey@medteq.jp
Post: 545-56 Tsujikuru-cho, Ise-shi, Mie, Japan 516-0046
Phone: +81 50 5532 9695